




OWNER'S OPERATING AND MAINTENANCE MANUAL

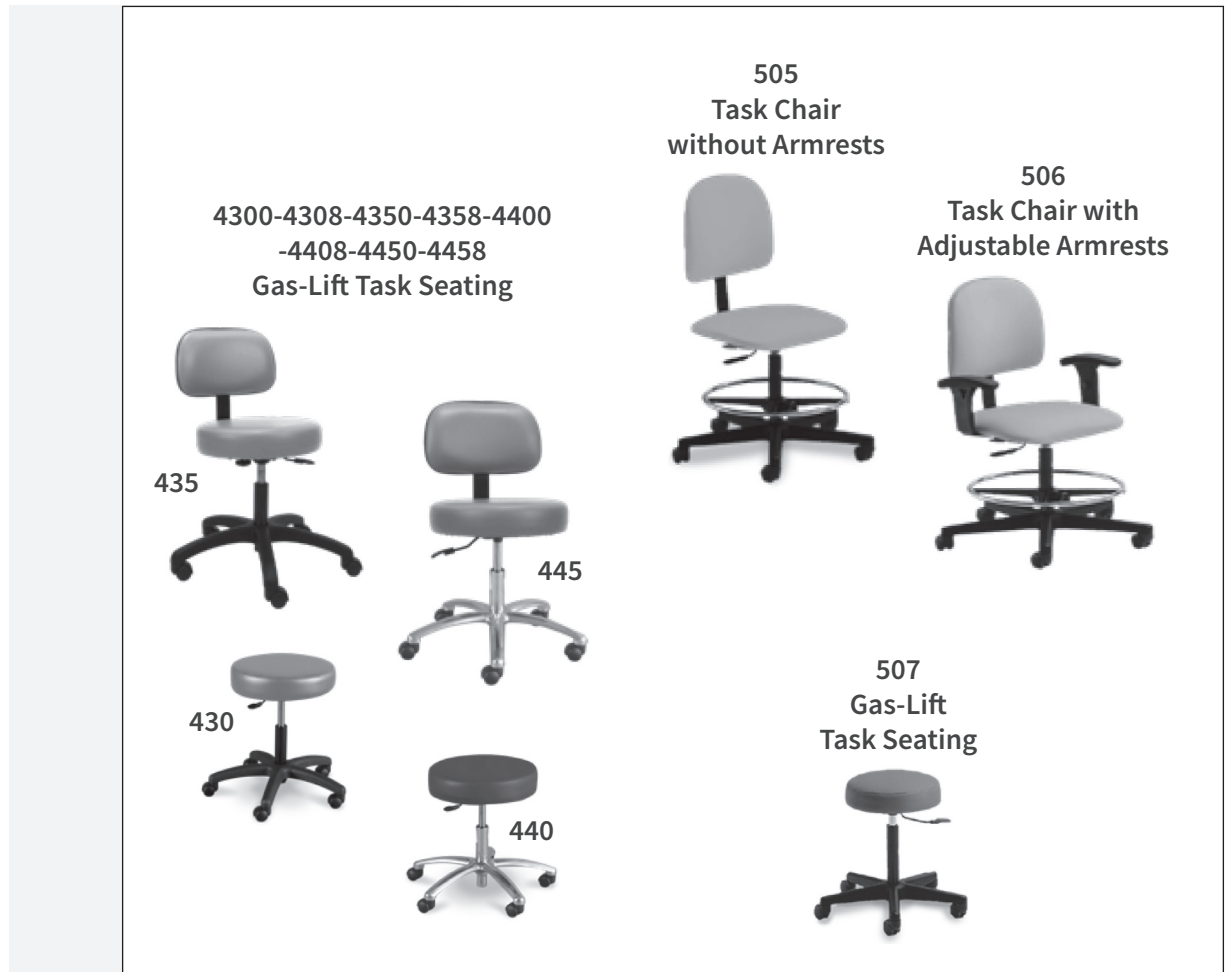

Task Seating

1-800-237-3377

CHAMPIONCHAIR.COM



IMPORTANT

READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT!

Do NOT install, maintain or operate this equipment without reading and following this manual, otherwise injury and/or damage may result.

IF YOU HAVE ANY QUESTIONS OR CONCERNS, PLEASE CONTACT CHAMPION MANUFACTURING, INC .

Champion assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.

No part of this manual may be duplicated in any form without the prior consent of Champion Manufacturing, Inc . Unauthorized duplication/distribution of these materials may result in civil prosecution to the maximum extent allowed by law.

The information contained in this manual is subject to change without notice.

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE!

These instructions are available online at no charge.
Visit ChampionChair.com to download.

TABLE OF CONTENTS

Important Safety Information 4

Transportation, Storage, Handling & Disposal 4

505 Task Chair without Armrests..... 5

506 Task Chair with Adjustable Armrests..... 5

 Base Assembly 5

 Back Assembly 5

 Arm Installation for 506 Model Only..... 6

 Operation of Task Chair 6

507 4300 4308 4350 4358 4400 4408 4450 4458 Gas-Lift Task Seating..... 8

 Base Assembly 8

 Back Assembly 8

 Operation of Gas Lift Stool..... 9

GENERAL CARE & CLEANING..... 10

Specifications..... 11



WARNING IMPORTANT: PLEASE READ

READ AND FOLLOW ALL DIRECTIONS.

1. **NEVER** stand on chair or **INJURY MAY OCCUR.**
2. **KEEP HANDS CLEAR** of moving parts when adjusting movable parts on chair or **INJURY MAY OCCUR.**
3. **DO NOT** use chair for transporting user in or with **ANY** type of vehicle of trailer. Chairs have not been tested or approved for use by an occupant in any type of vehicle or trailer.
4. **NEVER** leave an infant or child unattended on chair or **INJURY MAY OCCUR.**
5. **DO NOT** allow children to play on or with chair or **INJURY MAY OCCUR.**
6. Periodically, recheck tightness of all nuts, bolts, and screws.
7. **USE ONLY CHAMPION AUTHORIZED REPLACEMENT PARTS.**
8. **NEVER EXCEED** the recommended **weight capacity of:**
505 – 506 – 507: **300 lbs. (136 kg)**
4300 – 4308 – 4350 – 4358 – 4400 – 4408 – 4450 – 4458: **250 lbs. (113.4 kg)**
9. **SAVE THESE INSTRUCTIONS** for future reference and training.

TRANSPORTATION, STORAGE, HANDLING & DISPOSAL

- The product should be transported in factory packaging, inside an appropriate medium for the destination, i.e., air/sea cargo containers.
- The product should be stored in an environment that will inhibit rust/mold formation.
- Champion Manufacturing, Inc recommends not leaving the product in the factory packing in excess of three (3) months.
- The product should always be handled in a manner consistent with the user instructions and in a manner to prevent contamination after each use.
- The product has many recyclable components and to the extent practical, all effort should be used to recycle responsibly. Otherwise the product components shall be disposed of in accordance with local statutes.

NOTE: No tools are required for installation.

505 TASK CHAIR WITHOUT ARMRESTS

506 TASK CHAIR WITH ADJUSTABLE ARMRESTS

BASE ASSEMBLY

1. Place the **5-Star Chair Base** on a clean, flat surface with wheels down.
2. Push the large end of the **Gas Cylinder** into the center hole of the **5-Star Chair Base**. Press base and cylinder together firmly to secure (Fig. 1).








***Foot Ring Option** – Slide the **Foot Ring** over the **Gas Cylinder** and tighten the **Foot Ring Knob** (Fig. 2).

3. Install the **Chair Seat** over the end of the **Gas Cylinder**. Press Seat and Cylinder together firmly to secure (Fig. 3).

BACK ASSEMBLY

4. On the bottom of the **Seat Plate**, remove the **Knob** and **Hardware** (Fig. 4 & 7).
5. Slide the **Steel Back Support** of the **Upholstered Back** into the narrow slot on the rear of the **Seat Plate** and fasten with the **Knob** and **Hardware** (Fig. 5).
6. On the backside of the **Upholstered Back**, remove the **Knob** and **Hardware**. Insert the threaded stud of the back hinge through the slot on the **Steel Back Support**. Reinstall the **Knob** and **Hardware** to secure the **Upholstered Back**. Adjust the Backrest and Seat Knobs as needed for individual comfort (Fig. 6).

SEAT, BACK & BASE ASSEMBLY IS COMPLETE.

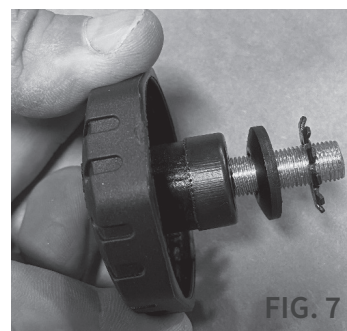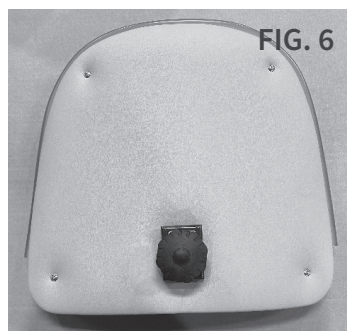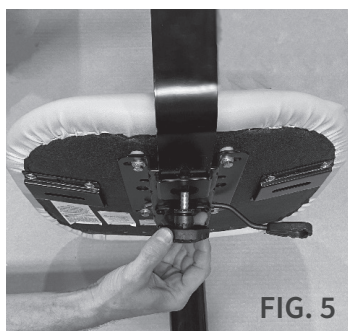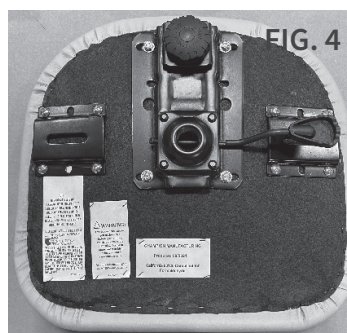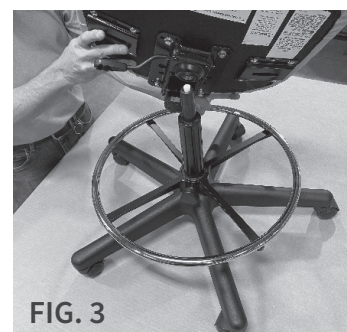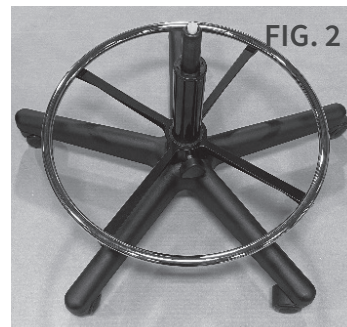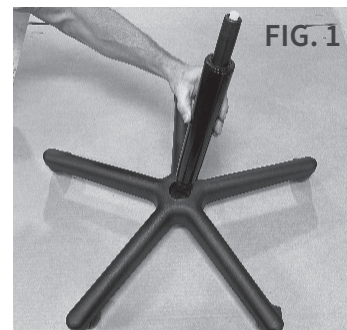

ARM INSTALLATION FOR 506 MODEL ONLY

7. Remove the **Arm Support Adjustment Knob** and **Hardware** from the **Adjustable Armrest**.
8. Slide the **Adjustable Armrest** into the **Armrest Bracket** located under each side of the **Upholstered Seat**. Secure the **Adjustable Armrest** by installing the **Arm Support Adjustment Knob** and **Hardware** through the slot in the **Adjustable Armrest** and into the threaded holes of the **mounting plate** (Fig. 8).

NOTE: The longer part of the arm should be closest to the front of the chair (Fig. 9).

9. Repeat for opposite side.

ARM INSTALLATION IS COMPLETE.

OPERATION OF TASK CHAIR

1. The chair is now ready for use. To raise the seating height, lift **Release Handle** without weight on seat.
2. To lower seating height, Lift the **Release Handle** while sitting on the seat; Release the handle when desired height is reached.
3. Position the **Adjustable Armrest** to desired height by pulling up on the **Arm Height Release Lever** and moving the **Adjustable Armrest** up or down. Release the lever to lock armrest in place.



WARNING

Periodically recheck tightness of all fasteners, and never stand on the Upholstered Seat.

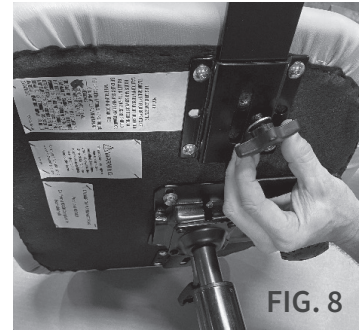


FIG. 8

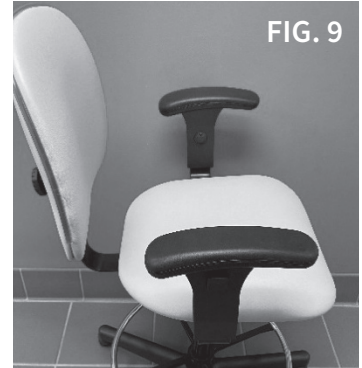


FIG. 9



IMPORTANT:

- **HIGH RESISTANCE (HR) CASTERS** – Your Champion task chair may be equipped with High Resistance (HR) casters. HR casters allow casters to roll freely when weighted or occupied but resist rolling when unweighted or when exiting the task chair. Standard casters always roll freely and may allow the task chair to move away from the occupant when leaving or entering the task chair. The potential for this unintended/unweighted movement may be reduced with the use of HR casters. Task chair users should be always attentive to their chair position when entering or exiting the chair.
- **NOTE: HIGH RESISTANCE (HR) CASTERS MAY NOT RELEASE AND ROLL FREELY IF YOU WEIGH LESS THAN 80 POUNDS.** High Resistance Casters look the same as standard casters and may not be easily distinguished from standard casters other than by loading and unloading them in a test. If your Champion task chair did not come with the HR Caster option, and you would like to add them, please contact Champion Mfg. for pricing and assembly instructions. Championchair.com (352) 854-2929.
- **WRINKLES IN VINYL** – Storage and shipping can cause wrinkles of the vinyl on your Champion task chair. This is normal. Please allow 1 – 2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.

507 GAS-LIFT TASK SEATING

4300 4308 4350 4358 4400 4408 4450 4458
GAS-LIFT TASK SEATING









BASE ASSEMBLY

1. Place the **5-Star Chair Base** (black plastic or chrome) on a clean, flat surface with bottom facing up (Fig. 1).
2. Install all (5) **Casters** into the **Base** (the stem of the caster should snap into place) and then flip the **Base** upright, with wheels facing down (Fig. 2 & 3).
3. Push the stem of the Gas Cylinder into the 5-Star Chair Base (Fig. 3).
4. Slide the **Foot Ring (optional)** over the **Gas Cylinder** and tighten the **Foot Ring Knob** (Fig. 4).
5. Install the **Stool Seat** over the end of the **Gas Cylinder** and push down firmly on stool seat (Fig. 5).

BACK ASSEMBLY (FOR MODELS WITH BACKS ONLY)

6. Slide the steel Slotted Back Strap into the Stool Swivel Plate (Fig. 6).
7. Insert the Back (depth) Adjusting Knob to lock the Slotted Back Strap into place (Fig. 7 & 8).

ASSEMBLY IS COMPLETE.

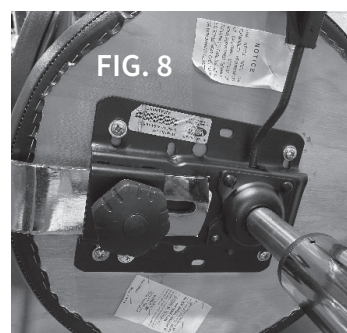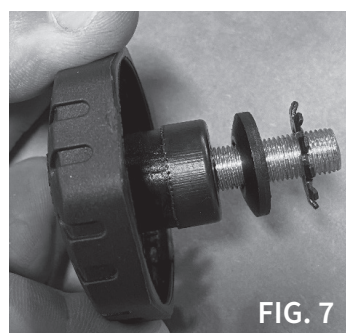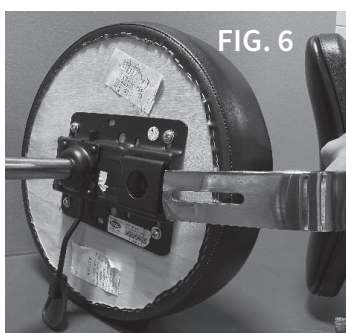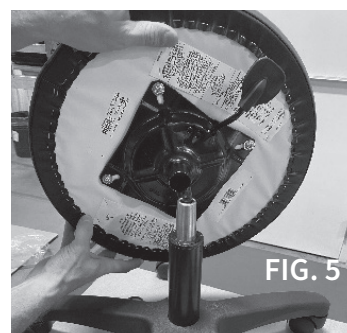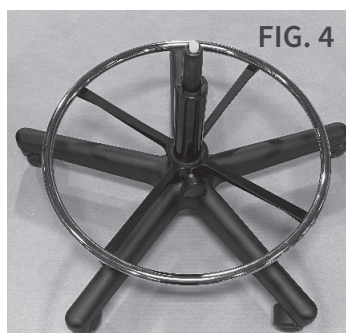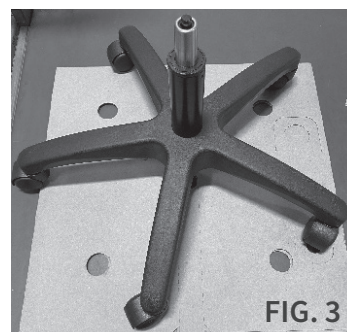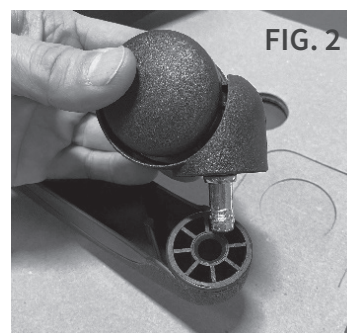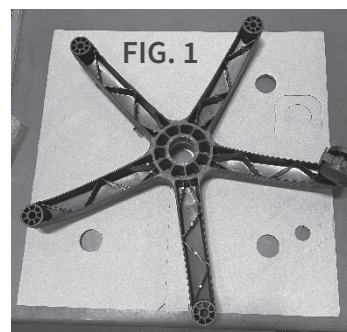

OPERATION OF GAS LIFT STOOL

1. The stool is now ready for use. To raise the stool seating height, lift **Release Handle** without weight on seat.
2. To lower stool seating height, Lift the **Release Handle** while sitting on the seat; Release the handle when desired height is reached. Approximate Range = 17 to 23 inches.
3. Position the optional **Upholstered Back** and **Foot Ring** into the desired positions using the **Adjusting Knobs**.



Periodically recheck tightness of all fasteners, and never stand on the Upholstered Seat.

IMPORTANT:

- **HIGH RESISTANCE (HR) CASTERS** – Your Champion task chair may be equipped with High Resistance (HR) casters. HR casters allow casters to roll freely when weighted or occupied but resist rolling when unweighted or when exiting the task chair. Standard casters always roll freely and may allow the task chair to move away from the occupant when leaving or entering the task chair. The potential for this unintended/unweighted movement may be reduced with the use of HR casters. Task chair users should be always attentive to their chair position when entering or exiting the chair.
- **NOTE: HIGH RESISTANCE (HR) CASTERS MAY NOT RELEASE AND ROLL FREELY IF YOU WEIGH LESS THAN 80 POUNDS.** High Resistance Casters look the same as standard casters and may not be easily distinguished from standard casters other than by loading and unloading them in a test. If your Champion task chair did not come with the HR Caster option, and you would like to add them, please contact Champion Mfg. for pricing and assembly instructions. Championchair.com (352) 854-2929.
- **WRINKLES IN VINYL** – Storage and shipping can cause wrinkles of the vinyl on your Champion task chair. This is normal. Please allow 1 – 2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.

GENERAL CARE & CLEANING

- Periodically inspect tightness of all screws, bolts, nuts, or other fasteners.

Champion furnishings are built to provide durable reliable service when properly cared for. In general all of our products should be: 1) Cleaned, 2) Rinsed and 3) Allowed to air dry. It is important to note that disinfecting a product is not the same as cleaning the product. Disinfectants alone will not provide adequate cleaning, they do not have grease or oil cutting properties to remove grime or hair and skin oils. Over time, the appearance and feel and performance of your upholstery may diminish if not cleaned properly.

CLEANING

Remove grime, hair, and body oils; Your furnishings should be cleaned with mild soap & water solution and a damp cloth on a regular basis. Avoid harsh detergents that could damage the finish of your product.

NOTE: If the furnishing is disinfected with bleach, it MUST be wiped off using only clean water on a damp cloth and then be allowed to air dry. Failure to rinse with clean water can result in a build-up of residues over time that can damage your product.

For additional cleaning information please visit www.championchair.com/cleaning-instructions/



IMPORTANT: ALWAYS REFER TO THE WEIGHT CAPACITY LABEL ON YOUR CHAIR.

In all cases the labeling on the chair at the time of delivery indicates the correct rating for your chair – Weight rating should not be exceeded!

SPECIFICATIONS



Task Stools	430	440	507	
Min. Seat Height	19" (48 cm)	17.75" (45.09 cm)	16.5" (41.91 cm)	
Max. Seat Height	21.5" (54.61 cm)	23" (58.42 cm)		
Overall Height	19" (48 cm)	17.75" (45.09 cm)	34" - 43"	
Seat Diameter/Width	16.5" (41.91 cm)	16.5" (41.91 cm)	15"	
Weight Capacity	250 lb (113.40 kg)	250 lb (113.40 kg)	300 lb.	
Footprint	22" (55.88 cm)	22" (55.88 cm)	22" (55.88 cm)	
Base Type High	Impact Plastic	Polished Aluminum	Composite Base	
Task Chairs	435	445	505	506
Min. Seat Height	17" (43.18 cm)	18.25" (46.36 cm)	21" (53.3 cm)	21" (53.3 cm)
Max. Seat Height	22.75" (57.79 cm)	23.25" (59.06 cm)	28" (71.1 cm)	28" (71.1 cm)
Seat Back Height	13" (33.02 cm)	13" (33.02 cm)		
Overall Height	29" - 34" (73.7 cm – 86.4 cm)	30" - 35" (76.2 cm – 88.9 cm)	37" – 46.5" (94 cm – 118.1 cm)	37" – 46.5" (94 cm – 118.1 cm)
Seat Diameter/Width	16.5" (41.91 cm)	16.5" (41.91 cm)	19"	19"
Weight Capacity	250 lb (113.40 kg)	250 lb (113.40 kg)	300 lb	300 lb
Footprint	22" (55.88 cm)	22" (55.88 cm)	26.5"	26.5"
Base Type	High Impact Plastic	Polished Aluminum	High Impact Plastic	High ImpactPlastic

IMPORTANT: Maximum Weight Capacity should be evenly distributed.

Visit our website ([ChampionManufacturing .com](http://ChampionManufacturing.com)) to view our full line of medical furnishings and procedure-chairs.

Additional copies of this manual and other product documents are available for download via the resources section of our website.

SPECIFICATIONS SUBJECT TO CHANGE

While we endeavor to maintain consistent products, due to conditions beyond our control, specifications are subject to change without notice. We reserve the right to add or delete products at any time and without prior notice. Always provide your product's specific serial number when ordering any replacement parts. When in doubt, contact a Champion representative (352-854-2929 or 1-800-237-3377) for further information.



Information contained in this document is based on the latest product data available at the time of printing. Photography, artwork, text and specifications are subject to change. Photographs may show optional items. Champion Manufacturing Inc. reserves the right to make changes to products, materials, options, and/or specification at any time without notice or obligation. Always provide your product's specific serial number when ordering replacement parts. Contact Champion direct for additional information and the latest specifications.